GA-21-1810-01  
Version 1.0  
Stand 01.03.2005

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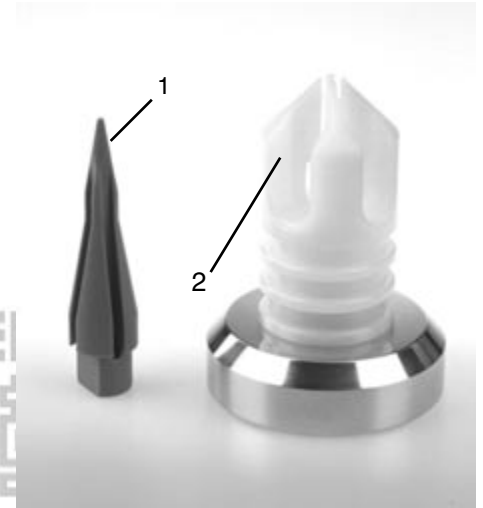
# SCHREIBER INSTRUMENTE

### **Operating instructions**

Take one of the rubber rings and push it on the top of the cone (1). Now take the cone at the bottom and push it slowly into the white counterpart (2). Take care that the your fingers of the counterpart are gliding along the corresponding grooves of the cone. Now the ring is at the shank of the cone. Take the loading cone (1) out of the counterpart (2) and plug the bottom of it into the head of your ligator. After that push the ring from the cone on your ligator which is now loaded and ready for use.

### **Pre-disinfection**

Immediately after using the device, immerse it in a disinfectant solution. This prevents general spread of bacteria and also helps to protect other hospital cleaning staff against contamination from bacterial secretion that may have adhered to the device and consequently present a health risk.

Immediate immersion in a suitable solution also prevents secretions from drying on the device, which makes it more difficult to clean off completely in subsequent cleaning.

When cleaning the device you must carefully follow the manufacturer's instructions on solution concentration and time taken to act, to ensure that the disinfectant has a chance to act thoroughly.

### **Cleaning and disinfection**

Clean the device immediately after pre-disinfection, with soapsuds and a soft brush. After cleaning, rinse thoroughly with distilled water and dry the device with a soft cloth. Do not sterilize with hot air or steam. Sterilize only by gas or disinfectant solution.

### **General instructions:**

- Always place the device on a solid base!
- Make Sure, that the diameter of the bottom of the cone fits the head of your ligator.
- Take special rubber rings which are very elastic and strong enough
- Store the rubber rings cool and dry. Do not expose the bands to the sun or halogen light for a longer period, otherwise they will become porous and easily burst.
- In the case your rings burst to often soak the bands in warm water before loading. During loading push the cone slowly into the counterpart. It may help to keep the band intact not loading it by one quick push but by two or three short and slow pushes.

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